

Seat heights and wheel positions

Each wheel combination has 100 mm height regulation. 70 mm adjusting the height bracket and an addional 30 mm by chaning wheel position.

With Netti V you have a total seat height range of 169 mm from 345 mm to 514 mm by changing wheel position and wheel size.

Wheel position bracket holes



Possible seat heights - from floor to top of seat plate.

Castors	Driving wheels	Wheel position	Possible seat heights
7"	16"	H1	
		H2	386-456 mm
5"	20"	H1	345-415 mm
		H2	
7"	22"	H1	
		H2	390-460 mm
7"	24"	H1	386-456 mm (standard) – see also page 2
		H2	416-486 mm – see also page 2
8"	24"	H1	
		H2	415-485 mm
8"	26"	H1	414-484 mm
		H2	444-514 mm (standard)



Adjustment of 70 mm using the height bracket



Adjustment of 30 mm changing the wheel position

Video tutorials:








Seat heights by adjusting the seat height bracket.

24+7" wheels

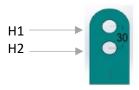

Bracket scale	Seat height (wheel position H1)	Seat height (wheel position H2)
0	386 mm	416 mm
1	396 mm	426 mm
2	406 mm	436 mm
3	416 mm	446 mm
4	426 mm	456 mm
5	436 mm	466 mm
6	446 mm	476 mm
7	456 mm	486 mm

